

400HR Horizontal Side Rails Instruction Manual

Alrick.

Product Code: 7002HR

Please Carefully read instructions before using the Rail.



INTRODUCTION

This instruction manual contains information on the instructions for use on operating this accessory.

These instructions for use may contain inaccuracies or printing errors. The information provided here will be updated periodically and changes due to product maintenance will be included in later editions. Changes or improvements are possible at any time without prior notice. If you have any questions, please contact our customer service team.

The instructions for use must be read and used by every person who operates this accessory.

PURPOSE

The intended purpose is to be used for individuals requiring support for mobilisation, repositioning and transfer requirements. Users must be instructed on its correct use and familiarise themselves with these instructions. Common users include nursing staff, physicians, therapists, family members, technicians, and cleaning staff.

PRE OPERATION CHECK

WITH RAILS IN POSITION, CHECK THE FOLLOWING:

- Make sure this product doesn't interfere with moving bed parts.
- When fitting safety sides to this bed please beware of the possible crushing points.
- The clamps can create crushing points on the rails.

SPECIFICATIONS

Overall Length - Folded (mm)	1830
Rail Width - Upright (mm)	1450
Height (mm)	412
Material Clamping Size (mm)	31.5 x 50.8
TGA Number	ARTG 217347
Manufacturer 	Active Medical, 104 Loughnan Street, Lake Cargelligo NSW 2672

IMPORTANT:

- **Warning:** The word Warning refers to a hazard to unsafe practice that may cause severe injury or death to you or another person
- **Accident risk Warning:** Do not use or allow use where the intended user has;
 1. Cognitive impairment
 2. History of recurrent falls
 3. Uses medication that would compromise falls
 4. Limited mobility comprises safe use
- **Note:** Designed to be fitted to Alrick products. Failure to comply may result in warranty being void.

FITTING THE RAILS

To fit the rails, use the below steps and the instruction video located on the bed (scan the QR Code).



STEP 1

Tools Required:

-



STEP 2

Place rails on correct side of the bed. (Release knob is always located towards the head of the bed.)



STEP 3

If required, change the width. Single width displayed above.



STEP 4

King single width displayed above.




STEP 5

Remove the mounts.




STEP 6

Placing the mounts. (Right hand side when laying on the bed)

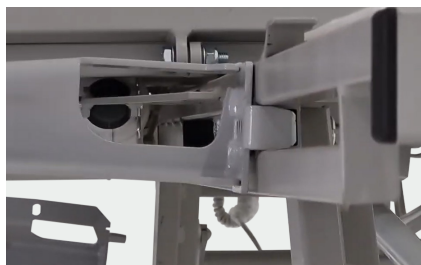

STEP 7

Rail placement on non actuator side.

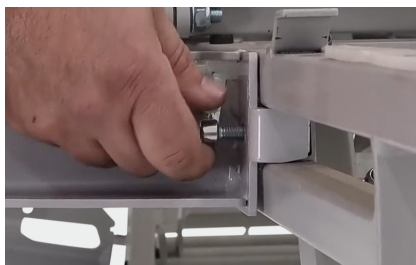

STEP 8

Fit the rail mounts at foot end of the bed.



STEP 9

Fit the rail mounts at head end of the bed.



STEP 10

Tighten all final bolts.



STEP 11

Remove kneebreak actuator on opposite side. (Left hand side when lying on the bed)



STEP 12

Slide in mount.



STEP 13

Loosen cable.



STEP 14

Locate rails in position.



STEP 15

Fit the rails in the mount.



STEP 16

Tighten all final bolts.



STEP 17

Refitting the actuator.



STEP 18

Finalise refitting the actuator.



STEP 19

Test rails before use.

MAINTENANCE

- Rails must be inspected regularly for any wear or movement that may cause injury, do not use if unsure.
- If one or more of the bed functions is found not working correctly, notify the manufacturer for advice.

EVERY 6 MONTHS:

- Check all accessories including rails are still functioning properly.

ANNUALLY:

- Professional service of product to be carried out annually & documented in 'Record of Performed Service Procedures and Regular Service Maintenance in the back of this instruction manual.

RECORD OF PERFORMED SERVICE PROCEDURES AND REGULAR SERVICE MAINTENANCE

Date	Purpose and Description	Performance

I confirm that technical staff have undertaken the repairs as described in the Record of Performed Service Procedures and Repairs Maintenance, and the equipment is now functioning properly.

Date _____ Customer Signature _____

WARRANTY

CONDITIONS

Your Alrick 400HR Horizontal Rails is guaranteed for **1 years** from the date of purchase. Professional service of product to be carried out annually and documented in the 'Record of Performed Service Procedures and Regular Service Maintenance. This warranty applies only on the parts manufactured or approved by Alrick.

EXCLUSIONS

This warranty does not extent those items that may need replacement due to normal wear and tear, or to damaged caused by misuse or accident, for which Alrick or it's distributors cannot be held responsible.

If your accessory should need attention due to failure as defined under the warranty above, please contact the distributor from whom you purchased your bed.

Please fill in the below and retain for your own records.

Date Purchased

Model

Serial Number(s)

Distributor

Distributor Address

Distributor Contact Number

RV 111225

Alrick.

1800 282 001
hello@alrick.au
www.alrick.au