

Secura 3 Mental Health Bed
Product Code: 7-SEC3

Instruction Manual

INTRODUCTION

This instruction manual contains information on the instructions for use and explanations of the signs and symbols used. These instructions for use contain instructions for operating the medical care bed.

These instructions for use may contain inaccuracies or printing errors. The information provided here will be updated periodically and changes due to product maintenance will be included in later editions. Changes or improvements are possible at any time without prior notice. If you have any questions, please contact our customer service team.

The instructions for use must be read and used by every person who operates the medical bed.

PURPOSE








The intended purpose of the Secura 3 Mental Health Bed is to be used for patients in hospital and healthcare facilities. These beds are electrically powered to allow the bed to be placed in various positions suitable to the patient. It is a medical device intended for human medical use only.

The Secura 3 Mental Health Bed moves on castors and features infinitely adjustable motorised height settings. The bedding area can be adjusted by multiple motors, allowing for positions such as sitting or exiting position.


The intended user of the Secura 3 Mental Health Bed is any individual who is being cared for in a mental health facility.

The bed should only be used by individuals trained or experienced in proper handling. Users must be instructed on its correct use and familiarize themselves with these instructions. Common users include nursing staff, physicians, therapists, family members, technicians, and cleaning staff.

SYMBOLS EXPLAINED

	Manufacturer
	For Indoor Use (House)
	Type B Application Part (Classification to the degree of protection against electric shock)
	Product with a Thermofuse
	Safety Isolating Transformer
	Device Protection Class II (classification according to the type protection against electric shock)
	Demko Approval

PRODUCT SERIAL
NUMBER PLATE

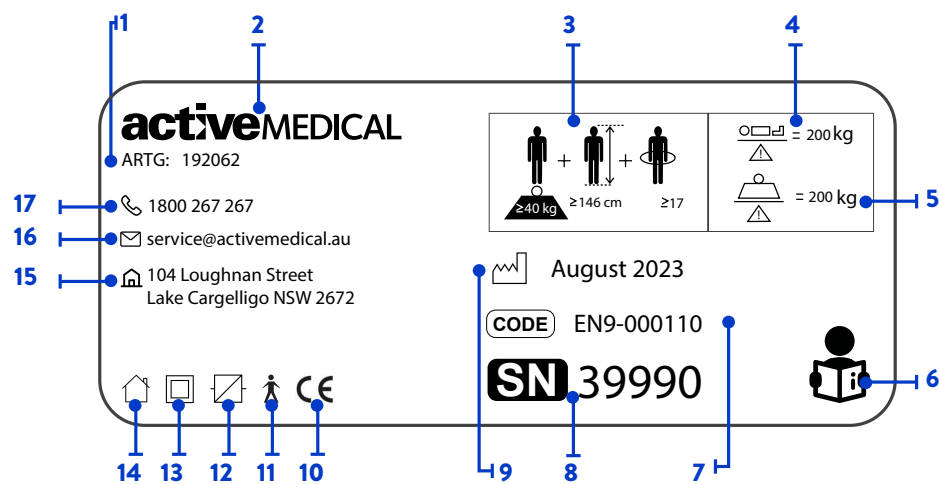

Serial Number Plates



SERIAL NUMBER
PLATE EXPLAINED

1	ARTG Number
2	Company Logo
3	Adult Identification (minimum weight, minimum height, minimum BMI)
4	Maximum Occupant Weight
5	Maximum Safe Working Load
6	Read Instruction Manual - Follow Instructions For Use
7	Product Code
8	Product Serial Number
9	Manufacture Date
10	CE Marking
11	Type B Application Part (classification according to the degree of protection against electric shock)
12	Switching Power Supply Unit
13	Device Protection Class II (classification according to the type protection against electric shock)
14	For Indoor Use Only
15	Manufacturer Address
16	Company Service Email Address
17	Company Phone Number

PRE-OPERATION CHECK

MODELS AVAILABLE

SPECIFICATIONS

With bed in position check the following:

- Electric beds should ONLY be used where earth leakage is fitted to the power source.
- Check power cord is plugged into 240V supply and is active.
- Check power cord is clear of castors and any moving part of bed.
- Ensure power cord has enough slack so cord is not strained when the bed is raised and lowered.
- Position bed at least 50mm from the wall in the lowest position. Lock castors. This position is critical so as not to damage the wall when raising the bed.
- Before using bed perform all functions to ensure they are working properly. If not please call your service technician or call the Active Medical Service Team on 1800 267 267

Secura 3 Mental Health Bed - Single Width







Secura 3 Mental Health Bed - King Single Width

Size	Single	King Single
Overall Dimensions	930 x 2030 mm	1100 x 2030 mm
Sleeping Surface	900 x 2000 mm	1070 x 2000 mm
Low Position	400 mm	400 mm
Weight Of Bed	140 kg	140 kg
High Position	700 mm	700 mm
SWL	250 kg	250 kg
TGA Number	ARTG 192062	
Manufacturer 	Active Medical, 104 Loughnan Street, Lake Cargelligo, NSW 2672	

ELECTRICAL DATA


U In	230C~,50Hz
I In	Max. 1.5A
U Out	24V... , max. 70W
Op.:	10%, Max. 2min./18min.
Protection Class	1Px4
Insulation Class	II Type B
Noise	48DBA

**** LOAD CAPACITY HAS BEEN TESTED IN ACCORDANCE WITH AS/NZS 3200.2.38:2007 ****

	For Indoor Use (House)		Equipment KI.2 (Double Square)
	Product with a Thermofuse		Demko Approval
	Safety Isolating Transformer	RW-Tüv approval	
	Type B equipment, as per EN60601-1		

IMPORTANT


- Check electric cords regularly and notify an electrician or Secura Service Dept on 1800 267 267 immediately. If any damage occurs cease to use and remove mains power until checked and repaired.
- No wiring is to be modified without authorisation from the manufacturer.
- Hi-Low model beds should be left in the lowest position when a nurse is not in attendance.
- Nurses, cleaners or anyone using Secura Electric Beds must make sure that the main power cord is clear of any moving parts on the bed before turning the power on.
- Electric Beds should ONLY be used where earth leakage is fitted to the power source.
- When patient condition could lead to patient entrapment the mattress support platform should be left in the flat position when unattended.

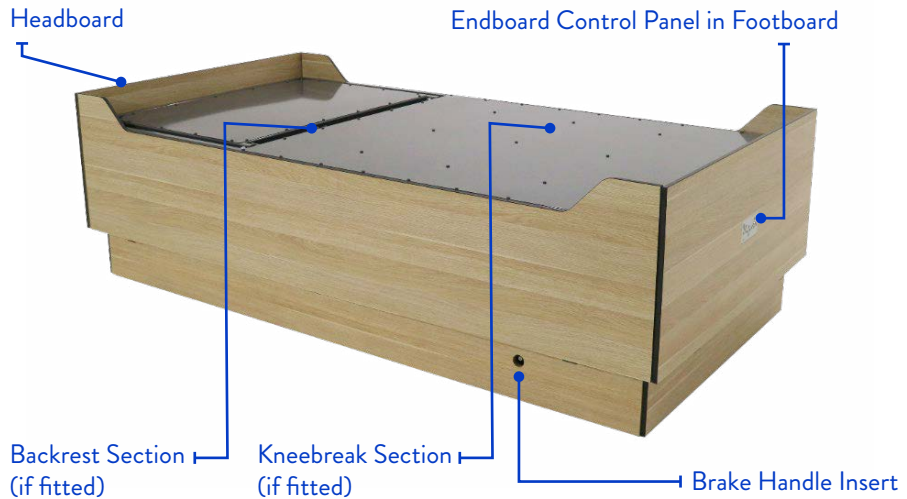

HANDSET FUNCTIONS



If operating the system for the first time or after an Actuator or Control Box replacement, the system must be initialised prior to normal use, please refer to the procedure below.

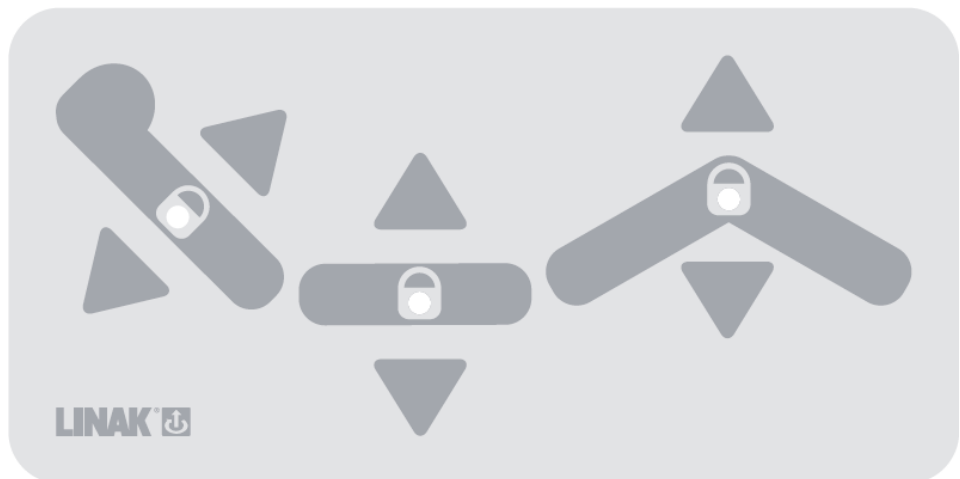

1. Press and hold '↑ BACKREST' will move CH2 'BACK REST' - OUT (if fitted)
2. Press and hold '↓ BACKREST' will move CH2 'BACK REST' - IN (if fitted)
3. Press and hold '↑ HILO' will move CH1 'HILO' - OUT
4. Press and hold '↓ HILO' will move CH1 'HILO' - IN
5. Press and hold '↑ LEGREST' will move CH4 'LEG REST' - OUT (if fitted)
6. Press and hold '↓ LEGREST' will move CH4 'LEG REST' - IN (if fitted)
7. To lock position press and hold both arrows at the same time

OPERATING THE BRAKING HANDLE

1. Raise bed to access the hole at the foot end.
2. Insert brake handle.
3. Rotate brake handle to foot end to engage the brakes. Rotate the opposite direction to release the brakes



SAFETY & WARNINGS

- Electric beds should **ONLY** be used where earth leakage is fitted to the room or building.
- If any cords are cut or damaged the bed **MUST** not be used until checked, fixed and tagged.
- No patient, service personnel or any person must lie underneath an Alrick bed while power is turned on, if not obeyed this may result in serious bodily injury or death if the bed controls are actuated.
- Please check no person or object is under the bed before using.
- Hand control to only be given to fully competent patients.
- Never leave hand control unattended with bed switched on unless patient is fully competent.
- Fit optional nurse lock out for safety of incompetent patients.
- Only one person is to be on the bed at any time.

MAINTENANCE

- Beds should be inspected monthly for abnormal wear. For example, loose or missing bolts, castors, rubbers or brakes damage.
- Rails must be inspected regularly for any wear or movement that may cause injury. Do not use if unsure.
- If one or more of the bed functions is found not working correctly, notify the manufacturer for advice.

EVERY 6 MONTHS:

- ✓ Check there is no visible damage to the electrical system or mechanical parts.
- ✓ Check wiring is secure and not interfering with bed movement.
- ✓ If actuator shafts are not clean – wipe chromed shaft of actuators to remove lint, grease and dirt.
- ✓ Extend actuators fully (do not wipe/wash as this will remove a fine layer of oil).
- ✓ Correct operation of all bed functions including castor locking.
- ✓ Check all accessories including rails are still functioning properly.
- ✓ Check all pivot points on the bed for wear. Replace pivot pins or bolts and removeable bushes (where fitted) if a pivot has a significant play.
- ✓ Check all functions still run smoothly.


Annually:

- Professional service of product to be carried out annually & documented in 'Record of Performed Service Procedures and Regular Service Maintenance in the back of this instruction manual.

CLEANING

- Cleaning can be done often – using domestic detergents.
- Do not use acids, alkaline or any other chemical solvents.
- When cleaning actuators, disconnect from mains power before wiping down with a moist cloth.
- When cleaning, be careful of entrapment, as it could cause physical injury.


MENTAL HEALTH ELECTRICAL ASSEMBLY AND TEST INSTRUCTIONS

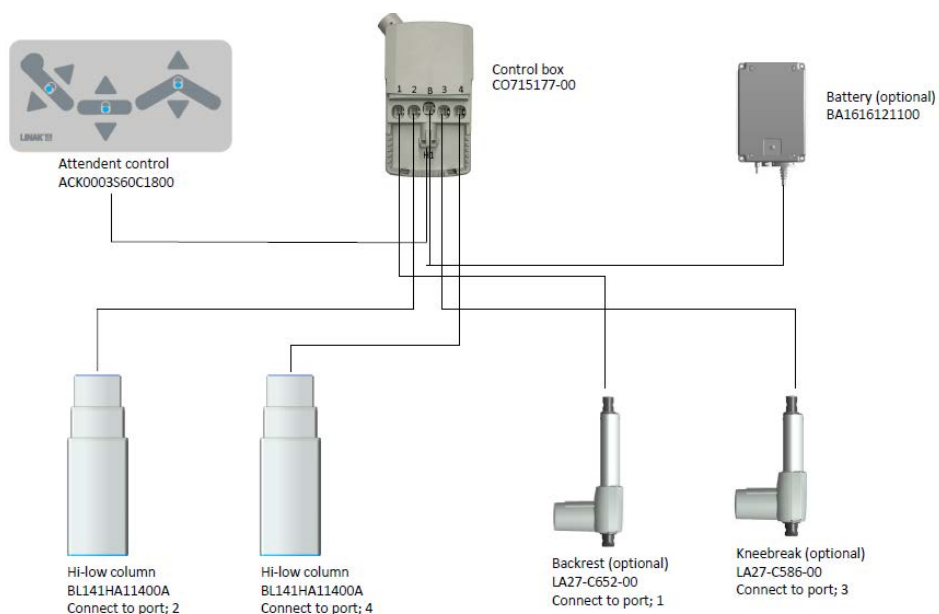

TROUBLE SHOOTING

Initialise

Press and hold the back up and bed up buttons at the same time until the lock LED light is flashing – then take the bed to the lowest position

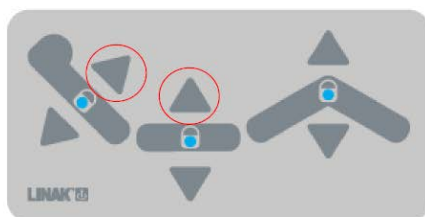

Fatal Error reset

Press and hold the back down and bed down buttons at the same time for five seconds



RECORD OF PERFORMED SERVICE PROCEDURES AND REGULAR SERVICE MAINTENANCE

[illegible]

[illegible]

I confirm that technical staff have undertaken the repairs as described in the Record of Performed Service Procedures and Repairs Maintenance, and the equipment is now functioning properly.

Customer Signature:

Date:

WARRANTY

CONDITIONS

Your Secura Electric Hi-Low Bed is guaranteed for 10 years on the frame from the date of purchase, 5 years on the electrics, 2 years on castors, 1 year on the battery.

Professional service of product to be carried out annually and documented in the 'Record of Performed Service Procedures and Regular Service Maintenance.

This warranty applies only on the parts manufactured or approved by Secura.

EXCLUSIONS

This warranty does not extend to those items that may need replacement due to normal wear and tear, or to damage caused by misuse or accident, for which Secura or its distributors cannot be held responsible.

If your bed should need attention due to failure as defined under the warranty above, please contact the distributor from whom you purchased your bed.

Please fill in and retain for your own records:

DATE PURCHASED:

MODEL:

SERIAL NUMBER/S:


DISTRIBUTOR:

PHONE:

EMAIL:

- E & O.E
- Specifications subject to change without notification

This image shows a single sheet of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

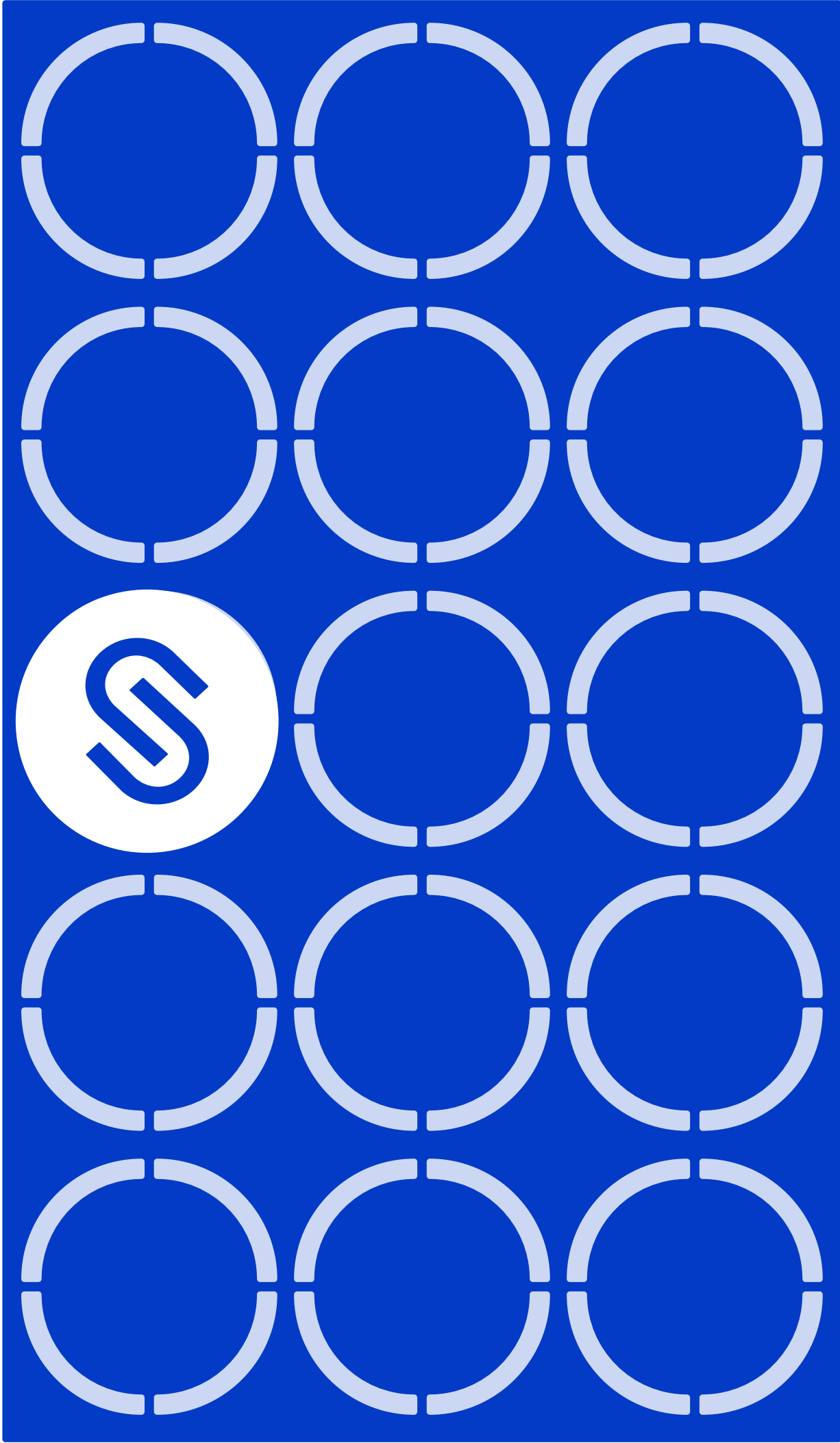

1800 267 267
www.secura.au