

Staebel Bariatric Patient Transfer User Manual

Product Code: 212147



Information before using the product / the functioning of medical products

1. Please read the enclosed instructions for use carefully.
2. Use the product only for the intended purpose of use.
3. Product management should be allowed only for the personnel familiar with the functioning of the product.
4. The maintenance as well as the safety and technical control must be carried out only by an authorised company or by any other professional company authorised by the Stäebel company.
5. Assembly, extensions, revisions or repairs must be carried out only by an authorised company or by any other professional company authorised by the Stäebel company.
6. In case of any technical interventions in the sense of additions or revisions as regards the products that are not carried out by Stäebel or any other professional company authorised by Stäebel, any guarantee in relation to revisions as well as in relation to the product and its functions referring to such revision, shall be abolished.

Preventive measures

Please read the instructions for use carefully. Use the product only for the intended purpose.

The remarks under the heading »WARNING« must be taken into consideration.

Intended use

Mover Flexi – Maxi is a mobile device designed especially to help a bariatric patient with a disability to be moved by a caregiver over short distances and to facilitate moving the patient from one sitting area to another. Mover Flexi – Maxi provides support to the patient when standing up from a bed, toilet, chair or a wheelchair. The product is intended for patients who are able to stand, but not necessarily walk.

Regulations and norms

Always ensure compliance with valid norms or regulations.

CE mark

CE mark refers to the Regulation for medicinal products MDR 2017/745.

Responsibility and guarantee

The manufacturer of the product is responsible for the safety and reliability of the product provided that:

A: The product is used solely for its intended use.

B: The functional check-ups are carried out on a regular basis.

Transport damages

Upon delivery, please, check the product and its packaging for damages. Always refer to the delivery date: visible damages should be reported within 2 days after delivery and hidden damages within 7 days after delivery.

Serial number

The serial number is located on the product label, which can be found on the inner side of the product left leg.


Product Display

1. Mover frame
2. Flexible base
3. Plastic footrest
4. Knee upholstery
5. Right seat upholstery
6. Left seat upholstery
7. Double-wheel caster
8. Double-wheel caster with brake
9. Maneuvering handle
10. Lever for adjusting flexible base
11. Two crossbars



Specifications

Product Code	212147
Width	740 mm
Length	980 mm
Height	1050 mm
Inside Base Width	490 - 670 - 860 mm
Outside Base Width	640 - 820 - 1010 mm
Weight	33.5 kg
SWL	220 kg

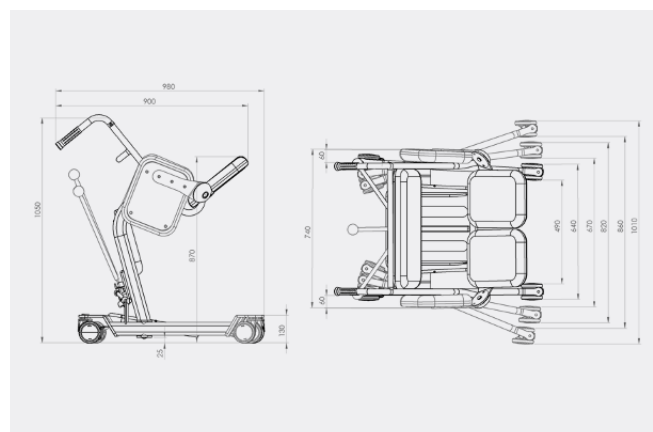

First use


The Staebel Bariatric Patient Transfer is ready for use immediately after removing the packaging.

Product use:

Adjust the flexible base of the Staebel Bariatric Patient Transfer with the lever and open up the two seat halves by manually pulling them on the side and pushing them upwards.



Step by step use

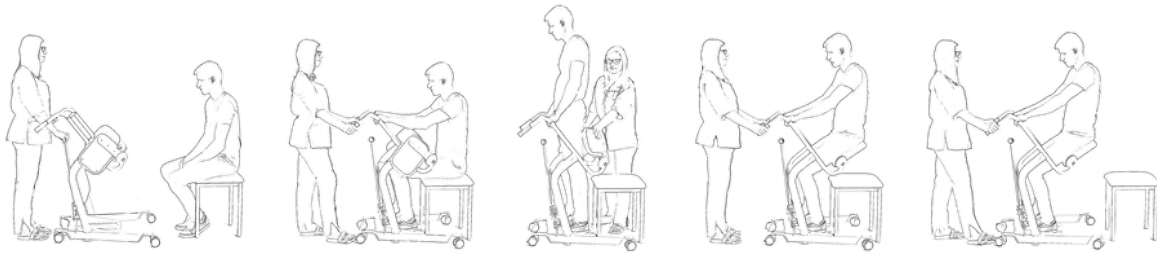

WARNING

- To prevent patients from falling during accommodating always apply the brakes before using the equipment. (See Fig. 1)
- To prevent patient falls or caregiver injuries make sure that there are two caregivers present during the transfer of a patient if this is necessary according to the patient state.
- To prevent damages to the equipment always close the padded seats. (See Fig. 2)



Fig. 1



Fig. 2

Maintenance

The product should be wiped with a duster. Approximately every two months the framework should be polished with a polishing paste. Knee and thigh supports are made of a material that is easy to maintain – composition leather, tested according to DIN 54020. Such coating may be wiped with a dry or damp cloth; detergent for sensitive laundry can also be used if necessary but it should be wiped straight after it has been used (the stains should be removed as soon as possible in order to prevent the absorption in the material). Finally, the material should be cleaned with warm and clean water.

Ingrained dirt can be removed if rubbed with a cloth, moistened with alcohol (the cloth should not be wet). Cleaning agents that contain fats or oils should not be used. Similarly, solvents (like acetone) or rough cleaning agents should not be used.

Safety and technical control

In accordance with »Medicinal Products Act« the safety and technical control should be carried out once a year.

Repair, servicing and spare parts

In case of any technical problems, please contact:

Staebel Products | 1800 772 287 | www.staebel.com.au | hello@staebel.com.au