

Staebel Transfer Platform User Manual

Product Code: 212150



Thank you for choosing Staebel Transfer Platform.

Read and follow all instructions, warnings and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. In addition, your safety depends upon you, as well as the carer or healthcare professional in using good judgement.

If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your authorised Staebel. Failure to follow the instructions, warnings and notes in this manual and those located on your Staebel product can result in personal injury or product damage and will void the Staebel product warranty.

Always follow the notes under the »WARNING« signs.

Intended use

Staebel Transfer Platform is a manually-operated device designed as a platform for use by healthcare professionals to assist in the physical moving (transfer) of a patient with a disability in an upright (standing) position over a short distance. The device consists of a wheeled platform, a surrounding framework, kneepad and handlebar. It can include safety belt, to hold the person being transferred in the standing position. Regulations and norms Always ensure compliance with valid norms or regulations.

Responsibility and Guarantee

The manufacturer of the product is responsible for the safety & reliability of the product provided that:

A: The product is used solely for its intended use.

B: The functional check-ups are carried out on a regular basis.

Transport Damages

Upon delivery, please, check the product and its packaging for damages. Always refer to the delivery date: visible damages should be reported within 2 days after delivery and hidden damages within 7 days after delivery.

Serial Number

The serial number is located on the product label, which can be found on the side of the plastic foot plate.

BEFORE USE

Information to read before use/How the medical product functions:

1. Before use carefully read the enclosed instructions for use.
2. Use the product only for its intended purpose.
3. Product can be used only by personnel who are familiar with the functioning of the product.
4. The maintenance, as well as the safety and technical check-ups, can be carried out only by an authorised service company or by any other professional company authorised by Staebel.
5. Assembly, revisions or repairs can be carried out only by an authorised service company or by any other professional company authorised by Staebel.
6. If technical modifications, including upgrades or revisions of the products, are not carried out by Staebel or any other professional service company authorised by Staebel, any guarantee for such revisions, as well as for products on which such revisions were carried out, shall be deemed void.

YOUR STAEBEL TRANSFER PLATFORM

Parts Designation

1. Handlebar
2. Kneepad
3. Plastic foot plate
4. Two wheels with brake
5. Three wheels without brake
6. Safety belt bracket
7. Central brake
8. Anti slip tape
9. Rising bars



Specifications

Product Code	212150
Width	610 mm
Length	670 mm
Height	1070 mm
Weight	16 kg
SWL	150 kg


First Use

The Staebel Transfer Platform is ready for use immediately after removing the packaging.

PRODUCT USE

Brake

1. Engage the brake:
Push the lever in the lower position to brake the casters.
2. Release the brake:
Lift the lever into the upper position to unlock the casters.
3. Safety belt (optional):
Place safety belt around the client's back and secure it with a clip.
When patient is in standing position attach the belt on the safety belt bracket.

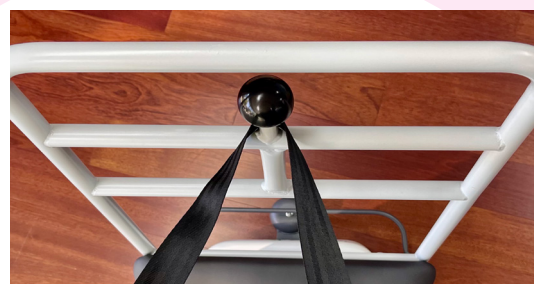

STEP-BY-STEP USE

- Always inform the patient about what you are planning to do.
- Position the Staebel Transfer Platform to allow the patient to place the feet on the foot plate with knees comfortably resting against the kneepad.
- Apply caster brakes.
- Help or encourage the patient to grab the rising bars and use it to stand up.
- Secure the user using the safety belt (optional).
- Release the brakes and proceed with the transfer.

Warnings:

- To prevent patients from falling during accommodating always apply the brakes before using the equipment (*See Image to the Right*).
- To avoid injuries, make sure that the patient is never left unsupervised.
- To prevent patient falls or caregiver injuries make sure that there are two caregivers present during the transfer of a patient if this is necessary according to the patient state.
- To prevent cross-contamination disinfect regularly between each use, if necessary, or daily, as a minimum.
- To avoid damage to the equipment, this product should exclusively be used indoors.



MAINTENANCE

The product is recommended to be cleaned and/or disinfected regularly between each use, if necessary, or daily, as a minimum. Clean the product with a damp cloth using warm water and a disinfectant cleaner. Disinfectant wipes, which are pre-soaked with a 70% (v/v) solution of isopropyl alcohol, can also be used. Knee support is made of composition leather, which is easy to maintain and complies with DIN 54020. This type of coating can be wiped with a dry or damp cloth; detergent for sensitive laundry can also be used, if necessary, but it must be wiped immediately after use (the stains should be removed as soon as possible to prevent its absorption by the material). Finally, the material should be wiped with warm and clean water.

Safety and Technical checks

Pursuant to the "Medical Product Act" the safety and technical checks should be carried out once a year.

Repairs, service and spare parts

In case of technical problems, please, contact Staebel or your local Staebel representative.