


Staeibel Zonecare 50 Medical Mattress User Manual

Product Code: 3M9050

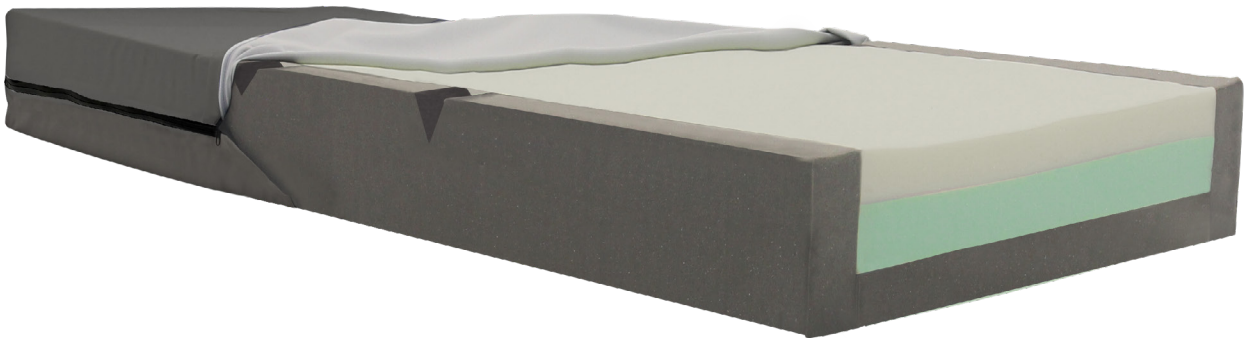

Thank you for choosing Staebel Zonecare 50 Medical Mattress

We aim to ensure you are satisfied with your mattress in terms of both performance and durability. We therefore kindly ask you to spend a few minutes reading through these user instructions to ensure you can take the best possible care of your mattress.

These instructions for use may contain inaccuracies or printing errors. The information provided here will be updated periodically and changes due to product maintenance will be included in later editions. Changes or improvements are possible at any time without prior notice. If you have any questions that are not answered in these user instructions, please contact us at Staebel. You will find our contact details on the user instructions.

- The product must only be used for its intended purpose.
- DO NOT USE in the vicinity of or in contact with fire sources/hot surfaces, such as fire, burning cigarettes or hot lamps. Although the product has undergone fire testing, it could become damaged if it comes into contact with fire sources.
- Position and use the product in a manner that prevents it from becoming trapped or damaged.
NOTE: Be particularly aware of trapping damage when using bed rails.
- Always make sure the mattress is the right size for the bed and patient.
- The cover does not allow liquid or air to penetrate but is vapour permeable.
- The mattress foam is protected by a cover; avoid using multiple hygiene covers as this can affect the vapor permeability of the mattress. It is also recommended to use a bedsheet on the mattress.
- Prevent damage to the cover from sharp objects.

CLEANING/DISINFECTION

Foam Core

- The hygiene cover protects the foam core completely, which means cleaning seldom is necessary. If, for any reason, the core needs to be cleaned, use a mild cleaning solution and water (for example spray on). Press out the water carefully.
- Let the foam core dry in a warm, ventilated area, without direct exposure of sunlight. The foam core needs to be completely dry before it is used again.

Mattress Cover



Note: Please consider other materials used in the cover construction as different conditions for cleaning will apply.

The mattress cover consists of PU and PVC materials. The mattress cover needs to be treated with care when cleaning the surface. Many industrial cleaning agents are extremely harsh on the coating and can cause it to break down, decreasing the life of the cover. The normal cleaning instructions are to wipe the surface clean with a soft sponge or cloth with warm soapy water. Mix with a mild concentration of detergent (1:20) or phenolic based commercial cleaner in warm water.

- Abrasive cleaning agents should not be used.
- The mattress cover can be washed in warm soapy water at up to 40°C.
- Do not use detergents containing solvent, peroxides or bleaches on the mattress cover without consultation with Staebel.
- For disinfection use a water/bleach solution of 10:1. Ensure the bleach does not have a sodium hypochlorite level higher than 5%.
- After applying detergent/disinfectant remove any residue with a clean, soft cloth and allow surface to air dry completely.

- Do not excessively rub or scrub and do not use abrasive implements to clean as these may damage the surface of the fabric.
- Mattress cover is not able to be autoclaved.
- Do not dry-clean.
- Do not tumble dry.
- Do not mangle.
- Do not iron.

STORAGE

Store in a cool dry area. Avoid excessive pressure and contact with non-colour fast materials. Make sure the mattress is stored where it will not be exposed to thrusts, hits or other things that can damage it.

SERVICE & WARRANTY

We warrant the product to be free from manufacturing defects for the period of **5 years** (foam core) and **1 years** (cover) from date of purchase. We will make available upon request; component part lists, descriptions, or other information that will assist service personnel to repair those parts of equipment that are designated by the manufacturer as repairable by service personnel.

Specifications									
Product Code	3M9050-EN	3M9050-N	3M9050-NINO	3M9050-S	3M9050-KS	3M9050-EW	3M9050-D	3M9050-Q	3M9050-K
Size	Extra Narrow	Narrow	Nino Cot	Single	King Single	Extra Wide	Double	Queen Double	King Double
Width	670 mm	720 mm	900 mm	870 mm	1050 mm	1150 mm	1350 mm	1480 mm	1780 mm
Length	1980 mm	1980 mm	1970 mm	1980 mm	1980 mm	1980 mm	1980 mm	1980 mm	1980 mm
Thickness	150 mm	150 mm	150 mm	150 mm	150 mm	150 mm	150 mm	150 mm	150 mm
SWL	150 kg	150 kg	150 kg	150 kg	150 kg	150 kg	150 kg	150 kg	150 kg